

Important – Please read carefully and keep this package insert for future reference.

# **Blink-N-Clean®**

## **Lens Drops**

---

**Improves lens-wearing comfort:** For soft and RGP contact lenses. Blink-N-Clean® Lens Drops is a sterile, isotonic, buffered, preserved solution.

**Contents:** Polyhexamethylene biguanide 0.0001% w/v as a preservative, purified water, sodium chloride, hydroxypropyl methylcellulose, tyloxapol and edetate disodium. This preparation contains no chlorhexidine, no thimerosal and no other mercury containing ingredients.

Blink-N-Clean® Lens Drops lubricates and rewets lenses, helps prevent protein film build-up, helps to remove particulate material that may cause irritation and/or discomfort. Use Blink-N-Clean® Lens Drops to promote lens cleanliness during wear, to rewet lenses before insertion and lubricate lenses during wear to moisten and reduce lens friction against the cornea. When wearing extended wear lenses, use Blink-N-Clean® Lens Drops to moisten lenses before retiring and upon awakening.

### **DIRECTIONS FOR USE**

To lubricate and rewet your lenses and to relieve minor irritation, discomfort, dryness, blurring and itchiness, apply one or two drops to each eye, up to four times per day, then blink several times.

If you require more frequent in-the-eye use of Blink-N-Clean® Lens Drops to maintain comfortable lens wear, this may signify a condition that should be evaluated by your eye care professional.

### **WARNINGS**

- To avoid contamination, do not touch the dropper tip of the bottle to any surface.
- Replace cap after using.
- Always keep bottle tightly closed.
- If irritation develops with the use of this product, discontinue use and consult your eye care professional.

### **PRECAUTIONS**

- Do not use if you are allergic to any of the declared ingredients.
- Keep bottle tightly closed when not in use.
- Store at 15° to 25° C.
- Use before the expiration date marked on the bottle and carton.
- Keep out of reach of children.
- Use before the expiration date marked on the bottle
- DISCARD SOLUTION 90 DAYS AFTER OPENING.

Questions or comments?

Please call us at 1-866-863-3873

[www.jjv-patents.com](http://www.jjv-patents.com)

Blink-N-Clean is a trademark of  
Johnson & Johnson Surgical Vision, Inc.

© Johnson & Johnson Surgical Vision, Inc. 2017  
Santa Ana, CA 92705, USA

AM70491EC12C

8772X

Revision Date: 12/2017